


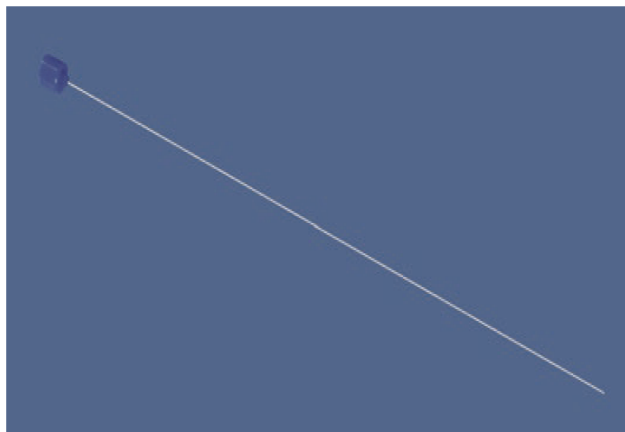

Not made with natural rubber latex

Description

The H/S Catheter Stylet is designed to gain ureteral access, to establish a tract, and to assist with device placement during gynecologic procedures.

Product Model

BA001

REF

Precaution

1. Do not force or over-manipulate the H/S Catheter Stylet when gaining access.
2. When using the H/S Catheter Stylet through a metal cannula, use caution, as damage to the outer surface may occur.
3. The H/S Catheter Stylet is not intended for PTCA use..

Caution

1. Sterile product unless package is opened or damaged.
2. Do not reuse to avoid the risk of infection caused by microorganisms.
3. This device is sold by or on the order of a physician.
4. Do not use the device if the sterile package is damaged or unintentionally opened before use.

Warnings

After use, dispose of the device as medical waste in accordance with applicable regulations.

Instructions for Use

1. Using aseptic technique, remove the H/S Catheter Stylet from its outer packaging and place it in the sterile field.
2. Remove the H/S Catheter Stylet from the holder.
3. Adjust the mandril position by advancing or withdrawing the mandril within the H/S Catheter Stylet to achieve the required degree of flexibility.
4. Advance the flexible end of the H/S Catheter Stylet into the HSG catheter and position it at the desired location.
Note: Advance the flexible end of the H/S Catheter Stylet into the HSG catheter and position it at the desired location.
5. Advance the flexible end of the H/S Catheter Stylet into the HSG catheter and position it at the desired location.

Glossary of Symbols

LOT	Batch code
	Manufacturer
MD	Medical device
	Temperature limit 10-40°C
	Humidity limitation 30-80%RH
	Biological Risks
	Do not re-use
	Consult instructions for use
	Expiry Date (YYYY/MM/DD)
	Caution
	Do not use if package is damaged and consult instructions for use
	Single sterile barrier system and using ethylene oxide
REF	Catalogue number
	Not made with Natural Rubber Latex

Reporting of Serious Incidents

If a serious incident related to the device occurs, immediately report the incident to Bioteque Taiwan Ltd. and to the applicable competent authority having jurisdiction in their locale.



Bioteque Taiwan Ltd.
7 F., No. 431, 433, Baozhang Rd., Xizhi Dist., New Taipei City 221036, Taiwan.
Info@bioteque.com