


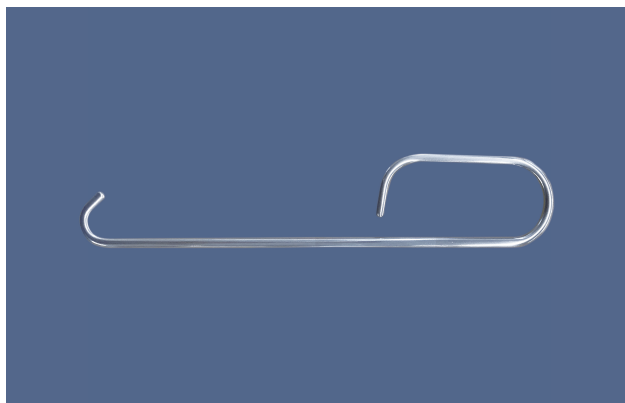

**Note: The Pessary Remover is used as a supplementary tool to remove the pessary and does not offer function of treatment.**

## Description

The Pessary Remover is designed for easy removal of the pessaries. It is best used for the removal of the ring, with or without support, the oval, dish, hodge or marland. When using the remover to remove the pessary, it must be used with lubricant.

## Warnings

1. Avoid the use of powerful force when inserting the device into the vagina. If the appearance shows discoloration, cracks or defects, stop using.
2. Dispose of medical waste properly in accordance with regulations after using.

## Instructions for Use

1. Lubricate the introitus, and hook remover inside on the center of the pessary (will not fit through hole, however).
2. Apply gentle traction on the pessary to stabilize and bring the device toward the introitus.
3. When the pessary is within reach, compress the pessary and carefully pull it gently out of the vagina.

## Cleaning

1. Prior to initial use, the Pessary Remover should be washed thoroughly.
2. Rinse Pessary with 20 - 40 °C warm water for 30-40 seconds.
3. Apply mild pH 5-9 soap to rub the pessary remover gently back and forth for 10 times to lather well.
4. Rinse off Pessary Remover with warm water for 1 minute, repeat and wash if needed.
5. Wash again and rinse Pessary Remover thoroughly with distilled water for 1 minute. Ensure all traces of soap are removed to avoid irritation.
6. Allow pessary Remover to completely air-dry before reinsertion.

## Glossary of Symbols

|                                                                                     |                                                                   |
|-------------------------------------------------------------------------------------|-------------------------------------------------------------------|
|  | Batch code                                                        |
|  | Manufacturer                                                      |
|  | Medical device                                                    |
|  | Temperature limit 10-40°C                                         |
|  | Humidity limitation 30-80%RH                                      |
|  | Consult instructions for use                                      |
|  | Expiry Date (YYYY/MM/DD)                                          |
|  | Caution                                                           |
|  | Do not use if package is damaged and consult instructions for use |
|  | Single patient multiple use                                       |
|  | Catalogue number                                                  |
|  | Biological Risks                                                  |
|  | Non-Sterile                                                       |

## Reporting of Serious Incidents

If a serious incident related to the device occurs, immediately report the incident to Bioteque Taiwan Ltd. and to the applicable competent authority having jurisdiction in their locale.



Bioteque Taiwan Ltd.  
7 F., No. 431, 433, Baozhang Rd., Xizhi Dist., New Taipei City 221036, Taiwan.  
Info@bioteque.com